


Diabetes Manager, general information:

This software is to be used by patients to analyze blood glucose and β -ketone measurement results. This data is neither shared, nor stored in the cloud. Languages: English and German. There are 2 versions: USB (compatible with all windows computers) and Bluetooth (compatible with windows 10 only). For a USB connection, you need a USB 2.0 cable of good quality.

Functionalities:

- A. Add new user
- B. Download data from the blood glucose meter (either by USB cable or Bluetooth)
- C. Personalize settings in the setup tab
- D. Data: analyze data
- E. Graph: analyze data
- F. Print: print analysis
- G. See contact details of the UK/German offices
- H. Delete user

Diabetes Manager: Desktop Icon

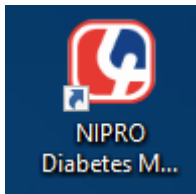

A. Diabetes Manager: Add new user

When you 1st use this software a pop-up will appear to enter your personal details. Please complete all fields and press next.

The screenshot shows the SURE Diabetes Manager software interface. At the top, there is a header with the SURE logo and 'DIABETES MANAGER' text. A 'User ID' dropdown menu is set to 'Lise'. Below the header, there are two red buttons: 'Add user' and 'Delete user'. A pop-up form is displayed in the center, containing the following fields:

- User ID:
- User Name:
- Birthday:
- Gender: Male Female
- Weight Unit:
- Height Unit:
- Language:

At the bottom of the form, there are two red buttons: 'Next' (circled in green) and 'Cancel'.

B. Diabetes Manager: Download Data

Click the download tab and indicate whether you are using a 4SURE Smart or Smart Duo meter. Connect your meter to your computer and then click next.

SURE | DIABETES MANAGER | User ID: Lise

Download

Step 1

The bluetooth version will say "via Bluetooth"

Connect the meter to your computer via USB cable. Ensure that the correct name is chosen in the User ID at the top of the screen.

Confirm the instrument and connection mode:

Instrument: 4SURE Smart Duo

Next

C. Diabetes Manager: Setup

- Personal information

Glucose target settings can be changed in the set up tab – personal information

SURE | DIABETES MANAGER | User ID: Lise

Setup

Personal information

User ID: Lise

User name: Lise

Date of birth: 05/03/2018

Gender: Male Female

Height: 0.0 cm

Weight: 0.0 kg



Target setting	Lower range	Upper range	Unit
Glucose	4.4	7.8	mmol/L

Target setting	Normal range	High range	Unit
β-ketone	< 0.6	≥ 0.6	mmol/L

Save


- **Instrument setting**

Tab to be used for date and time setting

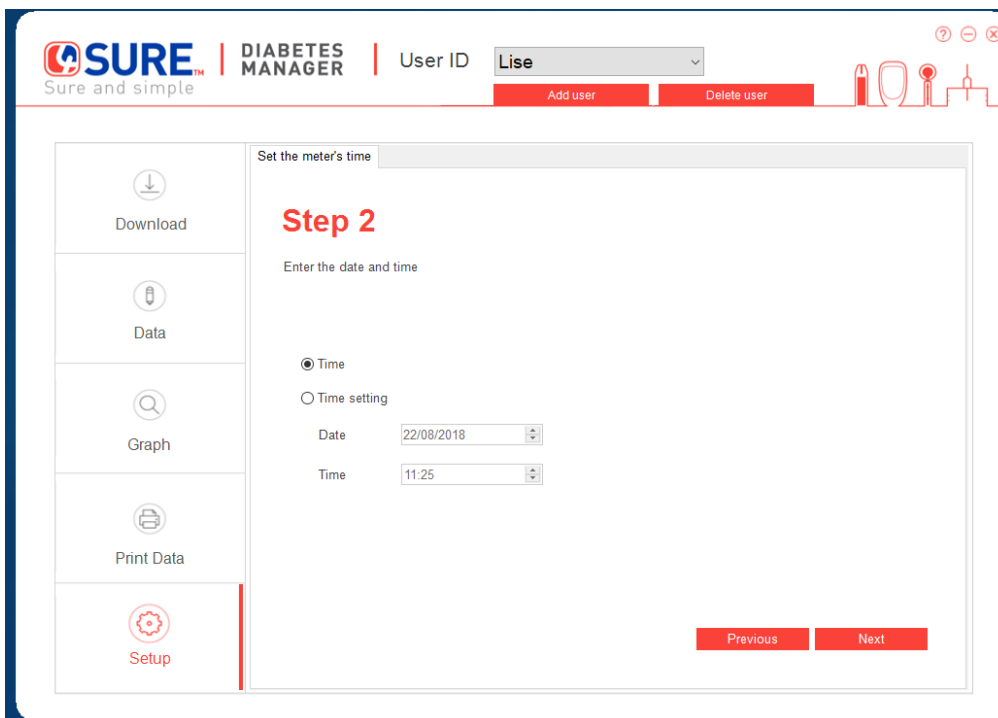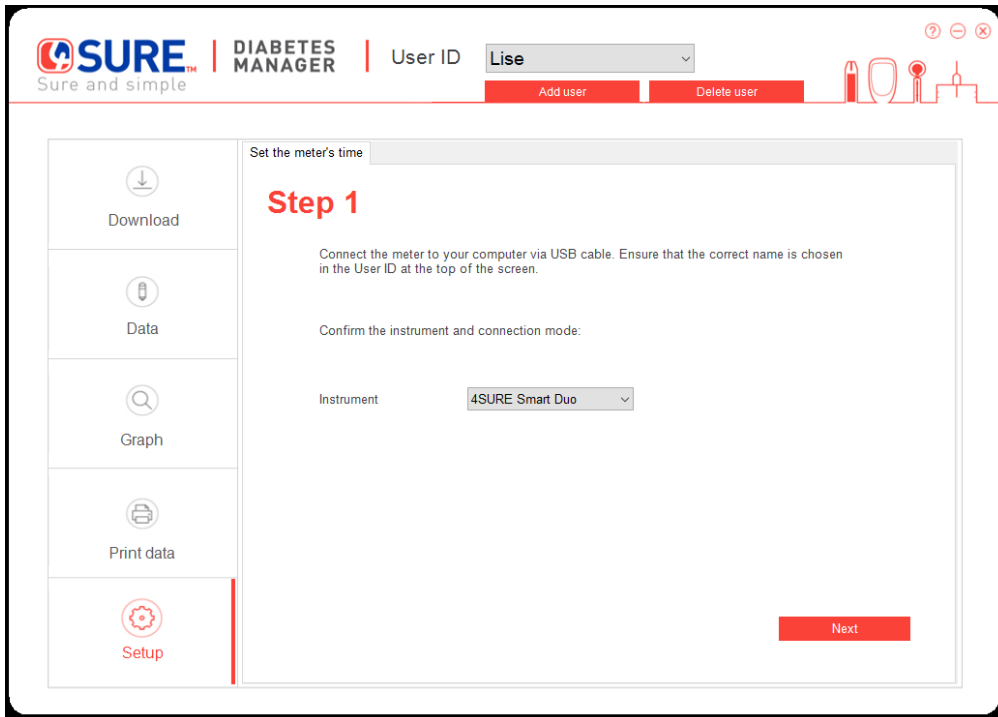


- Software setting

You can change language or units in this tab

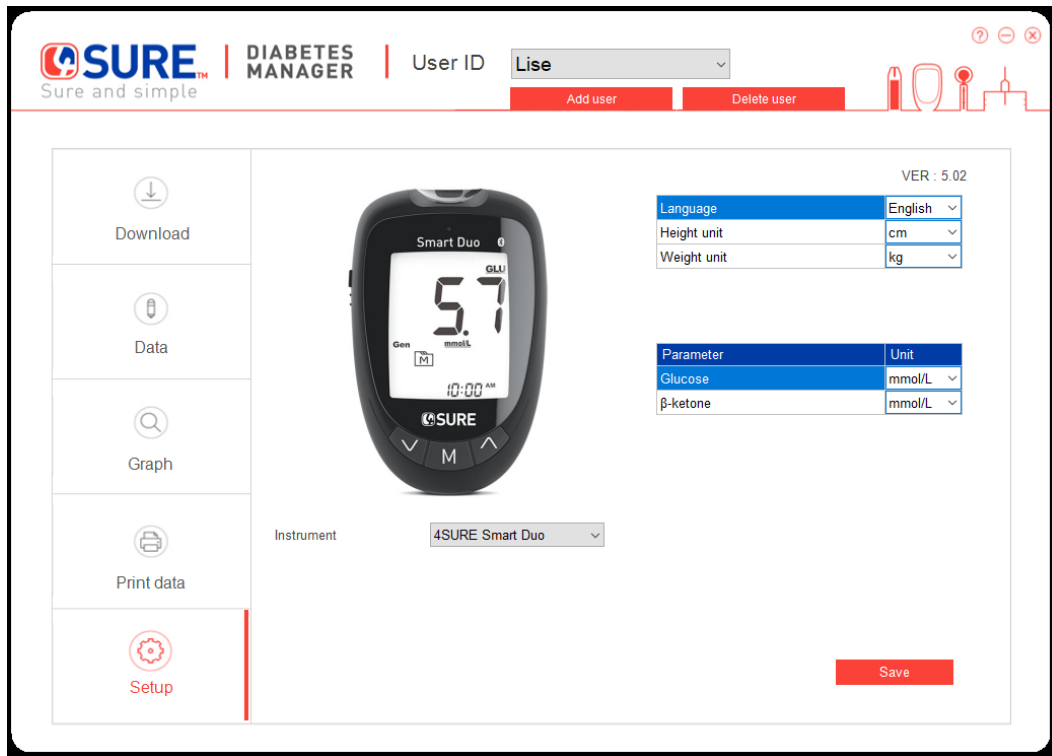


D. Diabetes Manager: Data

The data tab is where you can analyse your measurements. You can choose to see only blood glucose values, only β -ketone values or all values. The measuring mode defines whether you want to see only before meal measurements, after meal measurements or all.

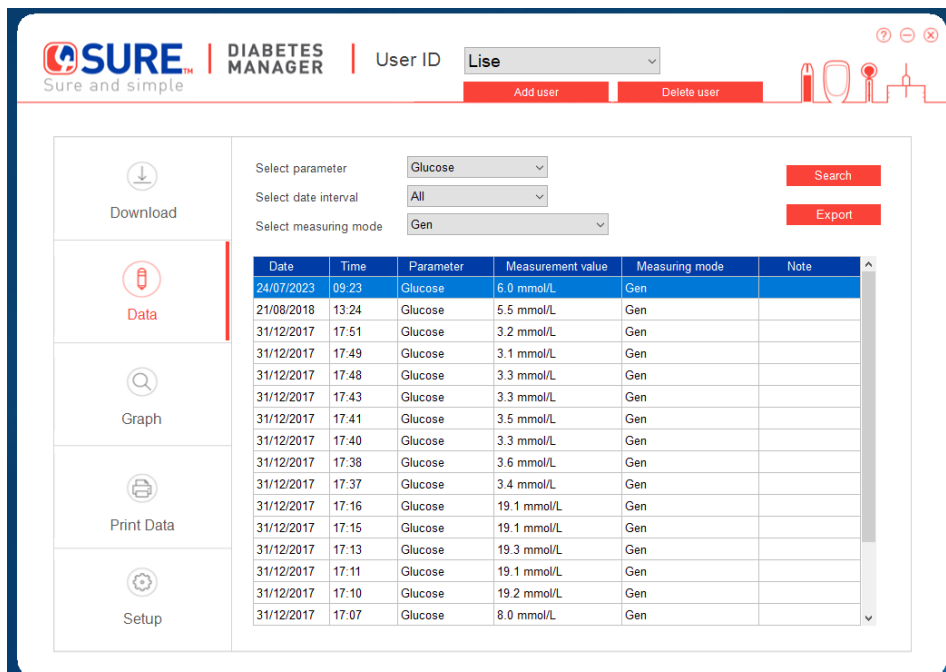


E. Diabetes Manager: Graph (visualizes the data in a graph)

The graph tab shows the same data as the “Data” tab, but visualizes it in a graph.

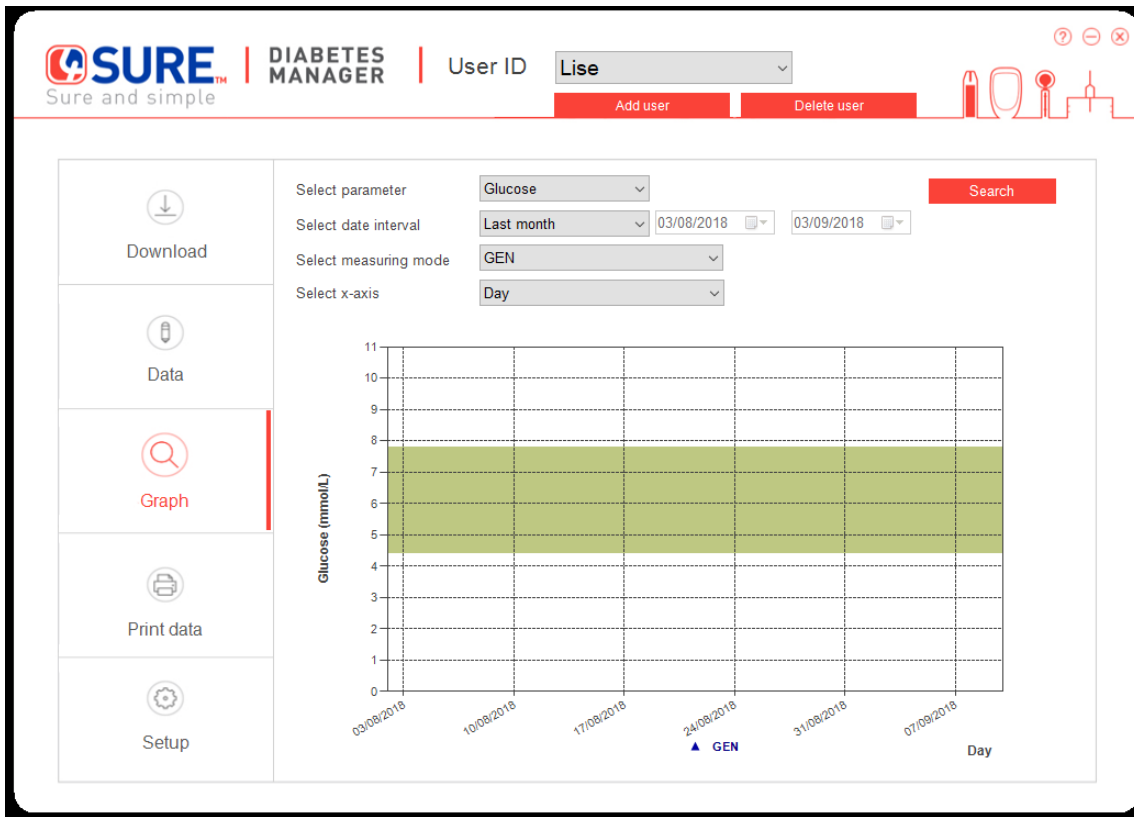


F. Diabetes Manager: Print data

You can use this tab to print your data. You will receive a PDF of the requested data for printing or for saving on your computer.

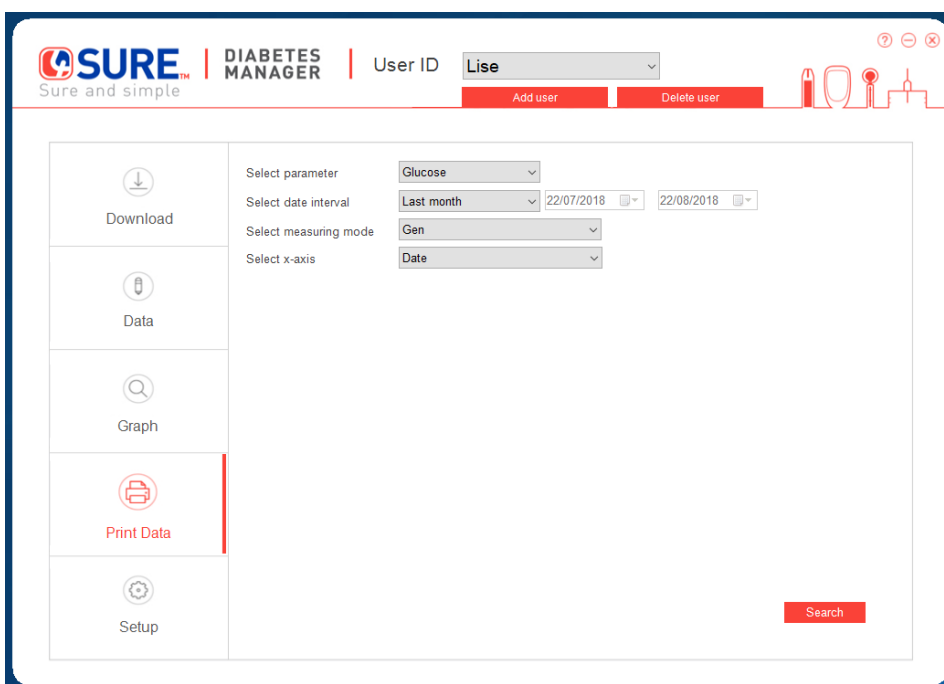

Example of PDF generated from 4SURE Diabetes Manager

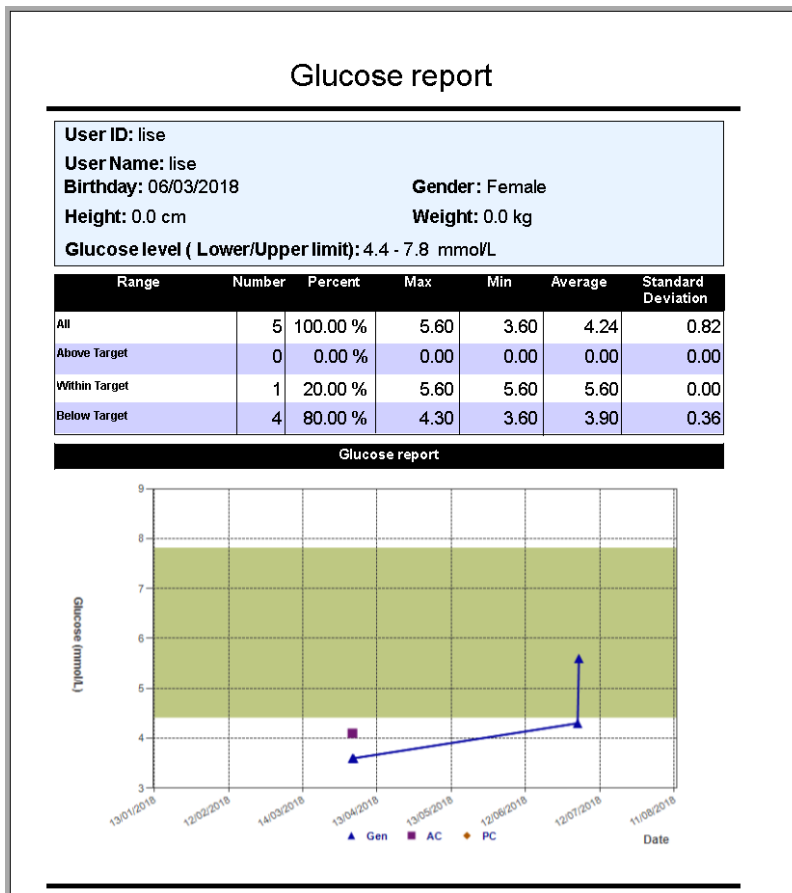

G. Diabetes Manager: Contact information

If you click on the question mark, you will see contact details of Nipro Diagnostics UK Ltd.

SURE | DIABETES MANAGER | User ID: Lise

Add user Delete user

Download

Data

Graph

Print Data

Setup

Select parameter: Glucose

Select date interval: Last month (22/07/2018 - 22/08/2018)

Select measuring mode: Gen

Select x-axis: [X]


SURE
Sure and simple

Nipro Diagnostics UK Ltd.
Units 12 - 14 South Point, Ensign Way - 5031 48F Hamble-Le-Rice,
Southampton, Hampshire - United Kingdom.
Customer Care: 0800 08 588 08
diagnostics-uk@nipro-group.com

Search

H. Delete user

You can at any time add or delete users.

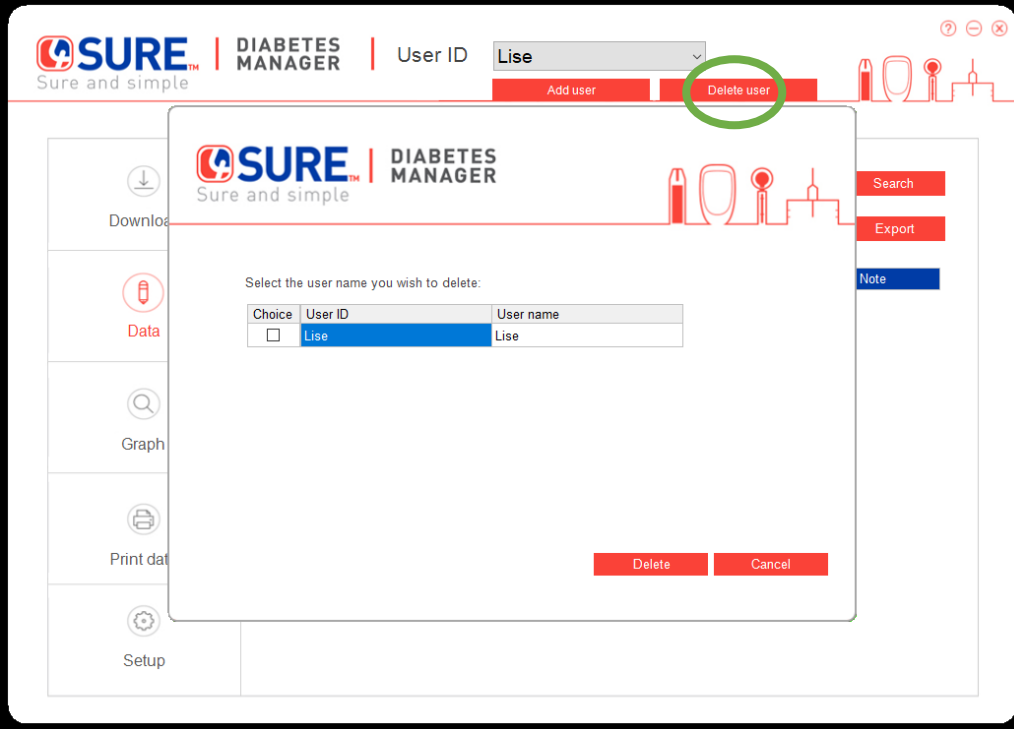

The screenshot displays the SURE Diabetes Manager web application. At the top, the logo 'SURE | DIABETES MANAGER' is visible, along with the tagline 'Sure and simple'. A dropdown menu for 'User ID' is set to 'Lise'. Below this, there are two buttons: 'Add user' and 'Delete user'. The 'Delete user' button is circled in green. A modal dialog box is open in the center, titled 'Select the user name you wish to delete:'. It contains a table with the following data:

Choice	User ID	User name
<input type="checkbox"/>	Lise	Lise

At the bottom of the dialog box, there are two buttons: 'Delete' and 'Cancel'. The background interface shows a sidebar with icons for 'Download', 'Data', 'Graph', 'Print data', and 'Setup'. On the right side, there are buttons for 'Search', 'Export', and 'Note'.